


## Downloading Intelviewer Software from North Coast Radiology Group Website

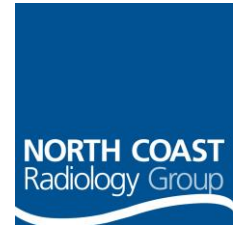

1 Web Location: [www.ncrg.com.au/referrer-support](http://www.ncrg.com.au/referrer-support)

2 Click on 'download software'

3 Within the software download section of website please select the vs of Intelviewer you need to install MAC, PC 32 or 64 bit.



[IV 32-bit Windows Installer](#)  
(password is referrer)



[IV 64-bit Windows Installer](#)  
(password is referrer)



[IV Mac OS X Installer](#)  
(password is referrer)



[Intelviewer Installation Guide](#)



[Intelviewer Quick Reference Guide](#)



[Intelviewer User Guide](#)

4 Click on the button to download the software. It will download to the 'downloads' location on your computer. This may take a few minutes. Once downloaded, open the attachment and follow the prompts. You may wish to check on the installation location of the software.

If installing a MAC version please be aware that you may need to change your system preferences security settings or it may not install correctly.

5 Once you have the icon on your desktop you will be asked to enter new server information; this is shown below.

Tick the protect bookmark and give a custom name you will recognise eg North Coast Radiology

